

Instructions for use (Korea)

1. Product Name: Surgical bone rasp
2. Model Name
BF1X, BF21, BF22, BF45, M-BF46, M-BF46A, FSH3-7, FSH5-11, PDS1-2S, PDS3-4S, PDS9-10S, BF1X-W
3. Purpose of use
A manually-operated rasp used to grind or cut the tissues during medical treatment.
4. Preparation
 - 1) Carefully check all product surface if there are any deformation, pin holes and foreign substance.
 - 2) Sterilization(Autoclave: Sterilization at 134℃ for at least 5min.)
 - ① Follow the manufacturer's sterilization method and operating instructions.
 - ② Place all instruments avoid contact each product surface by using sterilization wrap.
5. Method of use
 - 1) Follow the hospital manual for surgical bone rasp.
 - 2) If corrosion, cracks, etc. are found during use or dropped on the floor, immediately replace with another.
6. Storage and keeping conditions
After use, rinse with a cleaner or water to remove blood or foreign materials with a hand or a washing machine. Completely open the instrument having overlapping area before start the cleaning steps.
7. Cautions in use or precaution in use
 - 1) The sterilizer manufacturer's instructions should be followed in order to avoid staining when the instrument is sterilized. Keep the cover closed until the steam is completely removed.
 - 2) Do not use the product other than the intended purpose, store at room temperature.
8. Manufacturer information
Company name: OSUNG MND CO., LTD.
Company address: 57, Hwanggeum-ro 109 beon-gil, Yangchon-eup, Gimpo-si, Gyeonggi-do, Korea
Manufacturing site address: 57, Hwanggeum-ro 109 beon-gil, Yangchon-eup, Gimpo-si, Gyeonggi-do, Korea
Tel.: +82-31-987-5395
Fax.: +82-31-987-5397
Homepage: www.osung.co.kr
E-mail: qa@osung.co.kr

OSUNG MND CO., LTD
Authorized Signature:

Name and Title: Kumju Kwag / Quality Assurance Manager
Date of Signature: 2017.12.15.